


Basic Maintenance

STERLINK FPS-15sp

12 Month Maintenance



1 Year

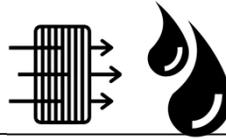

Filter & Oil Maintenance Kits



Change your **filters** and **oil** every 12 months or when indicated by color

Use Maintenance Kits to change **filters** and **oil**—Available for purchase from **VetOvation**


If you have any issues, send an email to **VetOvation**

Cary@vetovation.com

Maintenance Kits


Filters & Oil Change

Filter Kit

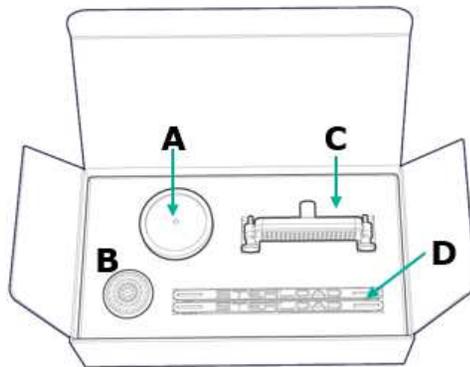


- A** HEPA Filter
- B** Chamber Filter
- C** Ozon Filter
(Two hand knobs)
- D** VC Cartridge

Oil Kit

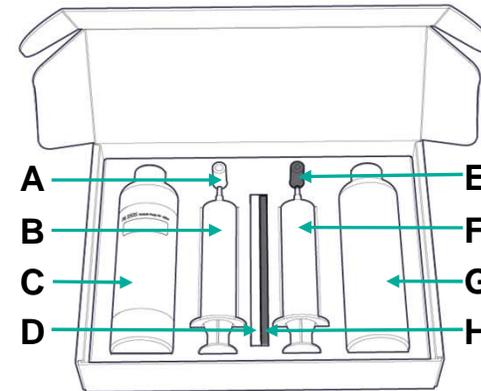


- A** New oil injection silicon adapter (white)
- B** New oil injection syringe
- C** New oil bottle
- D** New oil injection tube (transparent)
- E** Extracted oil injection silicon adapter (black)
- F** Extracted oil syringe
- G** Extracted oil bottle
- H** Waste oil extraction tube (black)

Oil Replacement Kit

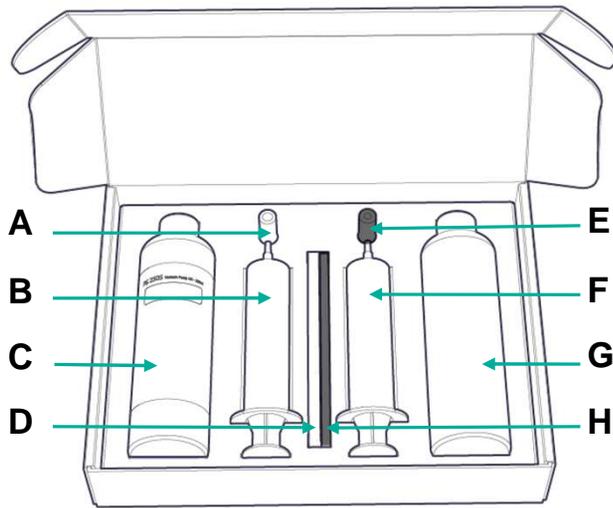

- A** New oil injection silicon adapter (white)
- B** New oil injection syringe
- C** New oil bottle
- D** New oil injection tube (transparent)
- E** Extracted oil injection silicon adapter (black)
- F** Extracted oil syringe
- G** Extracted oil bottle
- H** Waste oil extraction tube (black)

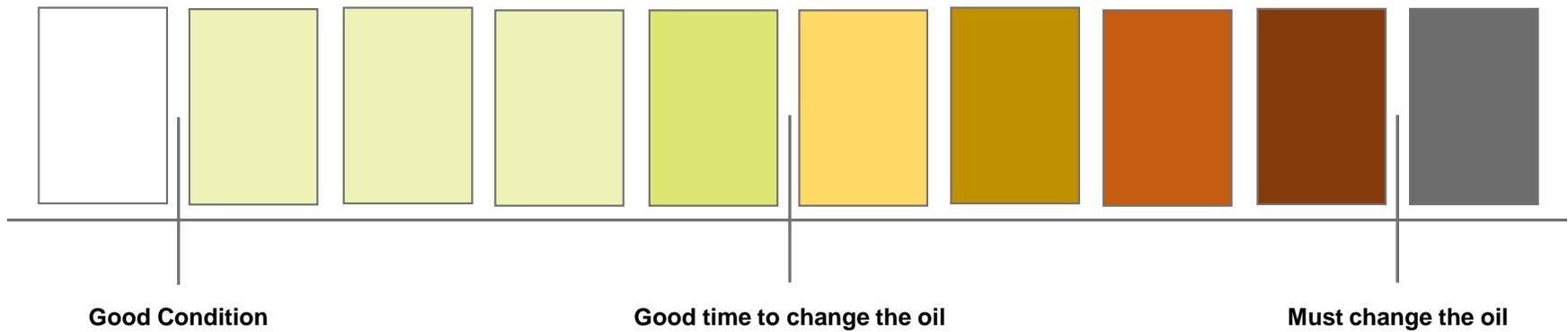

Oil Change Using Oil Replacement Kit



This is the oil changing kit containing 2 syringe, 2 tube, 2 tube connector, 1 oil 1 empty bottle



Assemble the rubber with the tube after that connect it to the syringe



Assemble the rubber with the tube after that connect it to the syringe



With your black tube connect it to the bottom blue hold from the unit.



After the tube fully connected slowly pull out the oil out from the unit.



With the empty bottle remove the tap.



Pick up the black tube syringe and push to put the old oil inside the bottle.



Repeat this until all the oil from the bottle is empty.




With a white tube syringe take out the new oil from the bottle.



The oil gauge should be slightly higher from the center just like the picture.

Filter Replacement Kit

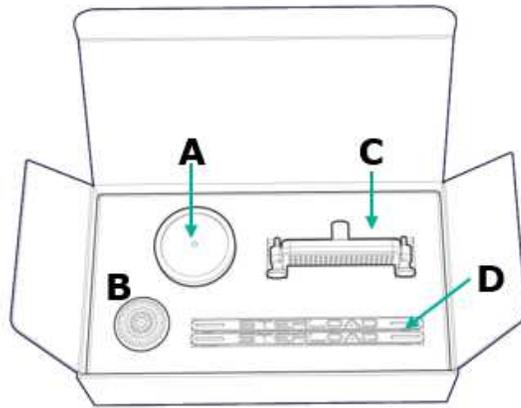

A

HEPA Filter

B


Chamber Filter

C

Ozon Filter
(Two hand knobs)

D

VC Cartridge

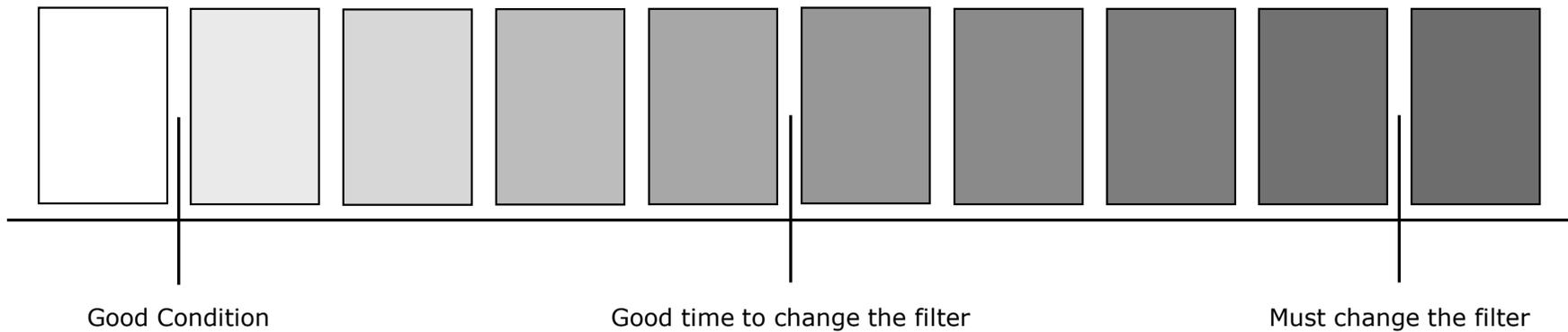

Good Condition

Good time to change the filter

Must change the filter

Filter Change Using Filter Replacement Kit

Changing HEPA Filter



1. Take out the new HEPA filter




2. Remove the old HEPA filter by turning counterclockwise




3. Replace it with new HEPA filter by turning clockwise

Changing Ozone Filter

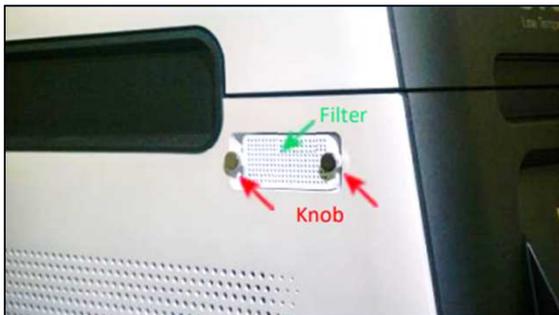

1. Ozone filter can be found on the left side of the product.

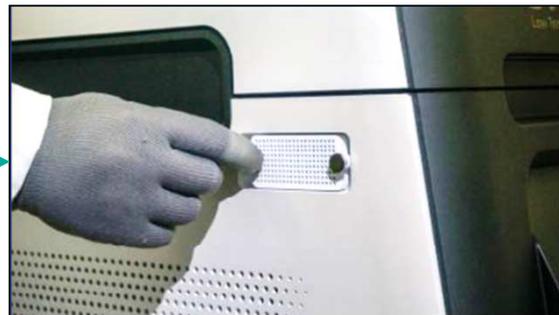

2. Remove the Knobs



3. Remove the Ozone filter and replace it with new Ozone filter

STERLINK Oil Replacement Kit

Product Name	Unit	Qty	Description / Specifications	Pictures
Oil Replacement Kit \$115 Order #: STER-OIL	Box	1	Regular Maintenance Kit	
Oil for Replacement	EA	1	Oil for Replacement LB100 (280ml)	
Waste Oil Bottle	EA	1	Waste Oil Bottle 300ml	
Oil Injection Syringe	EA	1	Oil Injection Syringe	
Oil Injection Tube (Clear)	EA	1	Oil Injection Tube – $\Phi 8$, 220mm	
Oil Injection Adaptor (White)	EA	1	Oil Injection Adaptor – White silicon	
Oil Extraction Syringe	EA	1	Oil Extraction Syringe	
Oil Extraction Tube (Black)	EA	1	Oil Extraction Tube – $\phi 6$, 220mm	
Oil Extraction Adaptor (Black)	EA	1	Oil Extraction Adaptor – Black silicon	

Available for purchase from:

vetovation

STERLINK Filter Replacement Kit

Product Name	Unit	Qty	Description / Specifications	Pictures
Filter Replacement Kit \$75 Order #: STER-FILTER	Box	1	Regular Maintenance Kit	
HEPA Filter	EA	1	HEPA Filter Storage temperature: +5~40°C	
Ozone Filter	EA	1	Ozon Filter Shelf life: None/Storage temperature: +5~80°C(Included Ozone Filter Hand Knob 2ea)	
Chamber Filter	EA	1	Chamber Hole Filter Shelf life: None Storage temperature: +5~80°C	
VC Cartridge	EA	2	VC Cartridge Shelf life: None / Not included w/hydrogen peroxide Required a calibration w/maintenance or when the installation place is changed	

Available for purchase from:

vetovation