



IntelliScope WiFi HD Camera System User Guide

For smart phone or tablet (Apple or Google)

Please call (919)247-0328 for assistance

1. Download the *EndoVision* App onto your phone or tablet




[DOWNLOAD *EndoVision*](#)



[DOWNLOAD *EndoVision*](#)

2. Connect the scope and light source to the WiFi HD camera handpiece

- Press the two tabs at the top of the camera coupler together (a)
- Insert the scope eyepiece into the coupler and release the tabs. The scope should now be seated securely in the coupler (b)
- Screw the portable LED light source onto the light post (c)

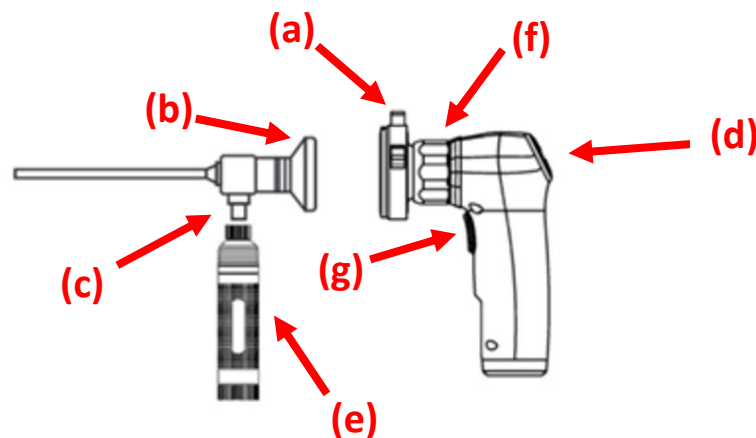

3. Turn on the IntelliScope System


- Turn on WiFi HD camera by depressing On/Off button (above the AWB button) for 3 seconds (d)
- Turn on portable LED light source (e) and verify it is on at scope tip
- Start *EndoVision* app on smart phone or tablet
- Confirm wireless connection with ENDOVISION_F6C2EC
- If needed, enter Password: **endovision88**

4. Scope Setup & Use

- White balance the camera by pointing the tip of the scope at a white paper or gauze then press & hold the AWB button (below camera On/Off button) for approximately 3 seconds. The screen will confirm when white balance parameters are set
- Focus scope in closed hand using black dial behind the coupler (f)
- Warm the scope in closed hand or warm water to prevent lens from fogging
- Watch screen as the scope is advanced into an ear to avoid collecting wax or debris on lens

5. Utilizing the App

- The app will orient the phone/tablet screen horizontally
- App Symbols (on left) include:

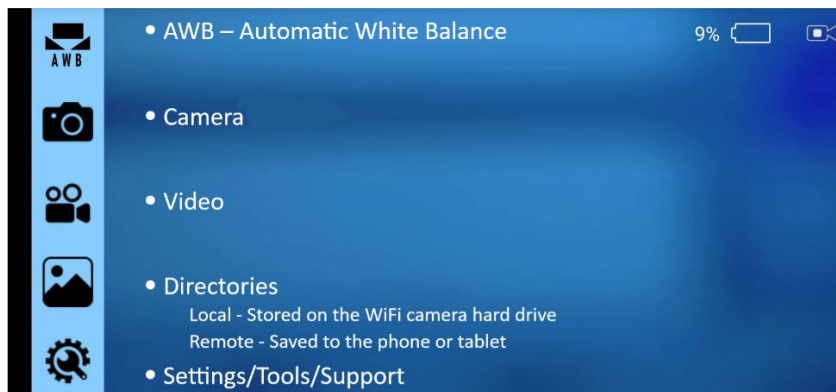

- To show/hide your icons and symbols, tap the screen at any time
- To close the app, swipe up on the lower white bar

6. Capturing Images & Video

- To capture **still images**:
 - Visualize target image on viewing screen and press & release the handle trigger (g) and the screen will display “Snap Successful”.
- To capture **videos**:
 - Visualize target then press & hold the trigger (g) for 2 seconds and the screen will display “Start Local Recording”. Note: Video camera icon turns red and flashes when recording.
 - To end video press & hold trigger button again and the screen will display “Finish Local Recording”. Note: Video camera icon returns to normal.
 - Still images may be captured while recording video by quickly depressing and releasing the trigger. The device screen will confirm image capture.
 - Still images and videos can also be initiated using the app if the device screen is touch activated. To capture a still of video simply tap the icons on the left of screen.

Thank you for purchasing the IntelliScope System by VetOvation

Please call (919)247-0328 for assistance if needed.